


InTouch Xpress™ Quick Reference Guide

Operating the Xpress



When the provider is logged in, their video feed and username is displayed. Otherwise, a screensaver will be seen on the screen.

Tap the screen (out of sessions) to access the toolbar, including Camera Preview, Settings, and Diagnostic Information (device status, battery charge, wireless signal quality, and other info). In-session controls are also available on the touch screen.

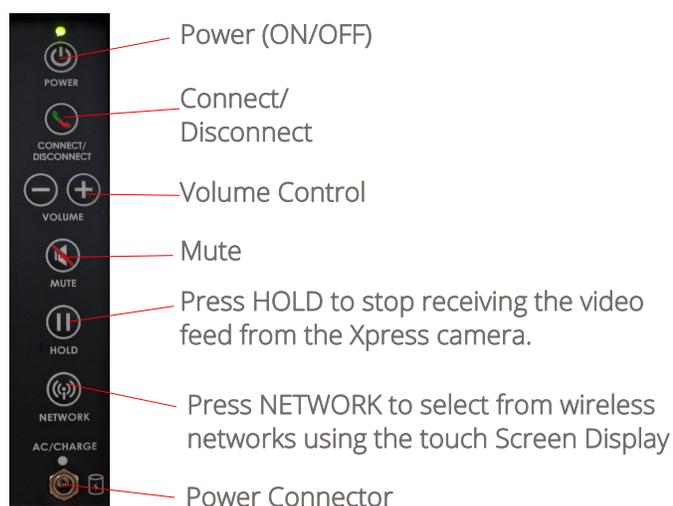

Installing the Battery and Charging

1. Slide the latch to unlock and open the battery door as shown.
2. Pull the tab to remove the battery.
3. Install the charged battery with the tab exposed.
4. Slide the latch to close the battery door.
5. Plug the Xpress AC/DC power adapter into a standard, grounded 120 Volt AC outlet to charge the device.

NOTE: The Xpress battery can be changed while in session if the Xpress is connected to an AC power supply with the InTouch Health provided power adapter.



Allow the Xpress to charge initially for at least 5 hours to reach a full charge. Other than charging the batteries, no user maintenance of the batteries is required.

The battery can last approximately 3 hours on a full charge. The Xpress will not run if the battery charge is less than 20%.

The system will charge to 20% from 85% in 4.25 hours.

The system will charge to 20% from 100% in 5 hours.

Xpress™ Quick Reference Guide

Xpress Accessories

- Xpress Mounting Arm, PN 20-14396
- Xpress Cart, PN 20-20195

Adjusting the Xpress Mounting Arm Accessory

1. Secure the arm clamp to an appropriate support.
2. Hold the platform (or Xpress handle) and unlock the Lock Release Lever to set the location of the mounting platform. The mounting platform should be parallel to the ground for best results.

CAUTION: When the Lock Release Lever is unlocked, all three pivot points are freed to move. Support the Xpress with its handle when repositioning so that it does not fall.

3. Lock the release lever.

WARNING: The Xpress mounting arm is adjusted at the factory to securely hold the Xpress steady under a vertical acceleration of 3 gravities with a moment arm of 10 inches (25.4 cm). To reduce the risk of injury, do not extend the mounting arm platform further than 10 inches (25.4 cm) from the base mounting point.

WARNING: To prevent the possibility of injury, do not secure the Xpress Mounting Arm to any support structure within 36 inches (91.4 cm) of a patient's head or critical areas.



NOTE: For best results, mount arm on flat, parallel surfaces.

Installing the Xpress on Mounting Platform

1. Insert the front edge of the Xpress mounting plate into the platform.
2. Lower the rear edge of the Xpress mounting plate on to the latch button. This will release the locking lever.
3. Press the locking lever tight.

