



InTouch Mini™




Wall Mounted




Mounting Guide

Hanging InTouch Mini Wall Mounted

Use these instructions to mount the InTouch Mini Wall Mounted to a predetermined location.



Step	Action	Image
1.	<ul style="list-style-type: none"> a. Use the Wall Mounting Plate as a guide to mark installation locations for the wall anchors. b. Use either the supplied anchors or other anchors deemed suitable by facilities personnel. c. Use the bubble level as a reference to ensure the Wall Mounting Plate is level prior to marking hole locations. d. Drill holes and install anchors. 	
2.	Fasten the Wall Mounting Plate to the anchors.	
3.	Position the Mini Wall Mount assembly over the Wall Mounting Plate.	

4.	Slide the Mini Wall Mount down over the Wall Mounting Plate until the locating pins on the top of the Mini Wall Mounting Plate are fully engaged.	
5.	Secure the Mini Wall Mount assembly to the Wall Mounting Plate using the supplied M3 Socket Head Cap screws.	
6.	Plug in the supplied AC cord into the bottom of the Mini Wall Mount assembly.	

Contact Information

24 /7 Live Technical Support

(877) 4 84 -9119

24 /7 Live Remote Technical Support & Live Chat

<https://intouchhealth.com/contact-us/>

E-mail Support

ITHSupport@intouchhealth.com

Website

www.InTouchHealth.com

InTouch Health User Manuals

<https://intouchhealth.com/manuals/>

Sales & Product Demos

(805) 562-8686



InTouch Technologies, Inc.

7402 Hollister Avenue, Goleta, CA 93117

Ph: 805.562.8686 • Fax: 805.562.8663

www.intouchhealth.com

