


InTouch Vici™ Quick Reference Guide

InTouch Vici At A Glance

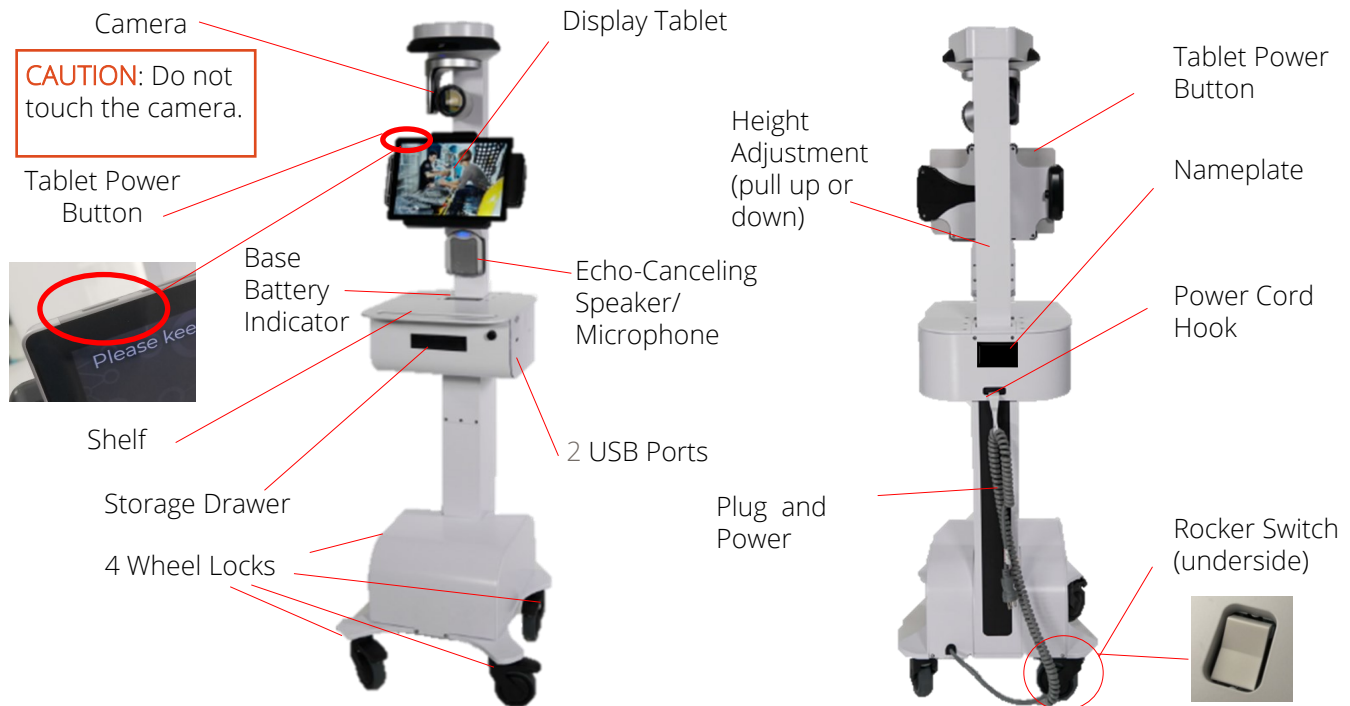


1. Power on InTouch Vici

NOTE: Refer to MA-20053 *InTouch Vici User Guide* if the Vici is not powered on.

Press the tablet power button to power on the InTouch Vici tablet.

- The software is pre-installed and will launch on startup.
- The screen to the right displays when tablet is on and the software initialized.

CAUTION: Keep the InTouch Vici device plugged in whenever possible to keep it fully charged and ready for use.

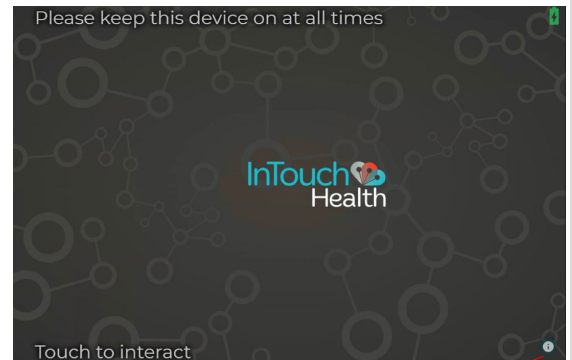

Displays Connectivity Level and Device Status.

2. Connect to Wi-Fi

- InTouch Vici Wi-Fi automatically connects to known Wi-Fi profiles.
- To change to another Wi-Fi profile, select **Wifi Setup** from the Settings screen and then select the desired Wireless SSID network. Contact InTouch Technical Services if needed.

3. Interaction

Interact with the Vici by tapping the touch screen anywhere to expose In and Out of Session controls. (Refer to MA-20053 *InTouch Vici User Guide* for more information.)

4. Request a Remote Consult

NOTE: You must use your standard protocol to request a connection from a remote provider such as a text or telephone request.

Once the remote provider has been notified to beam in for a consult, position the InTouch Vici in an orientation where the patient can see the provider's face on the tablet's screen.

InTouch Vici™ Quick Reference Guide

Adjust the tablet for easier viewing by tilting the monitor backward or forward.

CAUTION: Do not touch the camera.

Adjust the height by gripping the column and manually adjusting the column up or down (the position is held at any point on the column).

CAUTION: Vici should be powered on and charging whenever it is not in use



Adjusting Speaker Volume and Muting Microphone

Adjust the InTouch Vici speaker volume by pressing the **up** or **down** volume buttons on the Integrated Speaker Microphone.

Mute the microphone by pressing the **center** button.

- The lights on the device will flash when microphone is muted.

Moving the InTouch Vici

NOTE: Before moving, set the InTouch Vici at its lowest height, and unplug any attached auxiliary devices. Unplug the power cord and secure it on the power cord hook.

- I. Unlock the wheels before moving.
- II. Move the InTouch Vici to the desired location.
- III. Lock the wheels.
- IV. Plug in the power cord.

WARNINGS:

- Wheels may lock when InTouch Vici is in motion if lock is engaged accidentally.
- InTouch Vici may tip forward if pushed with too much force when wheels lock. **Please take care when moving to avoid damage or injury.**

